

# CATH•SECURE®


## Locking Multi-Purpose Tube Holder With Adhesive Tab *LOCK*

The CATH•SECURE® *LOCK* is the best option for securing all types of tubing.

With an added adhesive tab for stability, and a locking Velcro strip, the CATH•SECURE® *LOCK* provides superior securement without the need for plastic parts.

### Features and Benefits

- Adhesive strip to prevent rolling
- Cost effective
- No plastic parts
- Velcro strip to lock tubing
- Low profile
- Fits most types of tubing

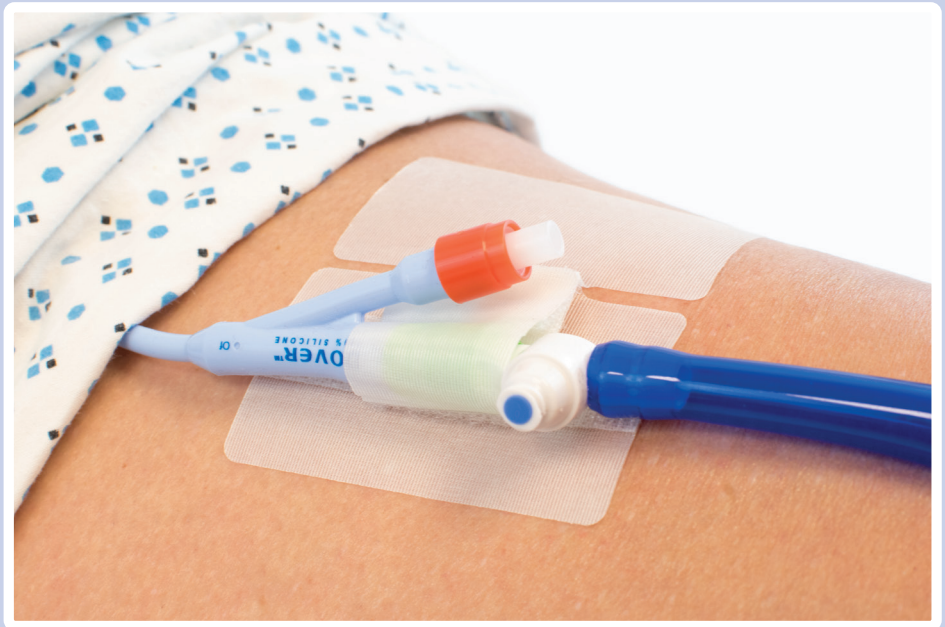

Scan to see the product in action



Email: [sales@mcjohnson.com](mailto:sales@mcjohnson.com)  
TO START YOUR FREE CLINICAL TRIAL

## APPLICATION



### Step One:

Peel back printed liner to expose adhesive.



### Step Two:

Place tube on exposed adhesive.



### Step Three:

Wrap the hook and loop tab around the tube.  
(Pinch any excess slack on the tab the tube keep from rolling)



### Step Four:

Remove the back liner and apply the device to the skin.



### Step Five:

Lock the secured tube to the velcro base.

# M. C. JOHNSON

M. C. Johnson Company, Inc.  
8801 Business Park Dr. Tel: (239) 433-0619  
Suite 201 Fax: (239) 433-1498  
Fort Myers, FL 33912 [sales@mcjohnson.com](mailto:sales@mcjohnson.com)